

Repair & Regenerate with DermeLuxx Pro

Combine the power of Diamond and Hydro-Dermabrasion to open skin channels and optimize delivery of Le Mieux growth factors to remodel skin and accelerate recovery without excess inflammation.

PROTOCOL SUPPLIES

- Peptide Foam Cleanser
- Skin Perfecter (optional)
- AML20 Peel Solution
- Zemits Hydro-Exfoliation Serum OR Iso-Cell Recovery Solution
- Ionized Oxygen Infuser
- EGF-DNA
- O² Calming Gel
- DermeLuxx Pro
- Hyaluronic Line-Filler Mask
- Bio Cell Rejuvenating Cream
- Just Glow BB Cream

Cleanse I	Lather a dime-size amount of Peptide Foam Cleanser with water. Perform cleanse, avoiding lash line, and remove.
Analysis	Identify client's skin concerns and assess skin barrier integrity. <i>Complete Client Intake Form and verify contraindications prior to treatment.</i>
Cleanse II	Perform second cleanse with skin-specific cleanser and remove. Optional: Use ultrasonic beauty tool, Skin Perfecter , to remove cleanser.
AML20 PEEL SOLUTION	Dispense 1-2 full droppers of AML20 Peel Solution into dish. Apply to skin with fan brush avoiding lips and orbital area. Gauge sensations. If sensations spike rapidly or become uncomfortable, remove immediately. Process no more than 10 minutes. Do not use steam. Remove with wet cotton until no sensations are felt.
DermeLuxx HydroDiamond™ Exfoliation	Optional: Apply 2-3 ml of Zemits Hydro-Exfoliation Serum, to skin with fan brush OR saturate skin thoroughly with Iso-Cell Recovery Solution during passes. Choose appropriate dual diamond tip. Perform HydroDiamond™ passes over solution.
Serum	Apply Le Mieux Repair & Regenerate blend in Ionized Oxygen Infuser : <ul style="list-style-type: none">• Load mini-flask with Iso-Cell Recovery Solution up to first line of flask.• Add 3-4 drops of EGF-DNA.• Swirl to mix, and mist close to skin until skin is fully saturated.
DermeLuxx Pro	Apply 2 pumps of O² Calming Gel with fan brush. Use constant circular motions with DermeLuxx Pro . Remove excess O² Calming Gel or leave on skin for increased benefit.
Jelly Mask	Mix 2-3 scoops Hyaluronic Line-Filler Mask with 1-2 scoops water in flex bowl. Use spatula to blend mixture into a smooth consistency and apply in downward strokes. Process until mask cures to a non-sticky consistency (up to 10 mins). Lift off mask and remove residue with steamed towel.
Finish	Moisturize and protect: <ul style="list-style-type: none">• Apply Bio Cell Rejuvenating Cream.• Finish with Just Glow BB Cream SPF50.