



ZEMITS DERMELUXX PRO:

Advanced Protocols with
HydroDiamond™ Technology

CONTENT

HydroDiamond™ Technology.....	03
Glass Skin Serums Collection.....	04
Zemits EquiSheer Pro.....	05
Zemits HydroDiamond™ Technology Tip Set	06
HydroDiamond Timeless Glow.....	07
HydroDiamond VitaFusion.....	09
HydroDiamond SkinDetox.....	11
HydroDiamond Luminance.....	13
HydroDiamond NeckTite.....	15
HydroDiamond ScalpBoost.....	17
Watch the Scalp Transformation.....	20
HydroDiamond SilkHands.....	21
Watch the Hands Transformation.....	24
HydroDiamond BackPurify.....	25
HydroDiamond Velvety.....	28
HydroDiamond SolarCare.....	30
HydroDiamond EvenTone.....	32
HydroDiamond CalmBreeze.....	34
HydroDiamond GentleCare.....	36
HydroDiamond Red Carpet	38
HydroDiamond GentleM.....	40
HydroDiamond ClearTeen.....	42
HydroDiamond DePuff.....	44
HydroDiamond GlowMama Safe & Soothing Facial Treatment Protocol.....	46
Watch and Learn: Step-by-Step HydroDiamond Facial Training.....	49

HydroDiamond™ Technology:

RESULTS EVERYONE WANTS

What is HydroDiamond™ Technology?

Zemits HydroDiamond™ Technology is a next level of evolution of HydroDermabrasion. It combines the power of Diamond and Hydro-Dermabrasion in a single, seamless treatment. This innovative approach allows for both deep exfoliation and thorough extractions, delivering exceptional results for a variety of skin concerns in just one session.

Why spa-owners love HydroDiamond™ Technology?

Zemits HydroDiamond™ maximizes treatment effectiveness and profitability by merging two powerful modalities. It saves time, drastically cuts down product expenses, and delivers noticeable results after just one session.



THE ZEMITS ADVANTAGE



With Zemits HydroDiamond™ Technology:

One bottle of Hydro Serum lasts up to 100 facials.



Price per Treatment Average: \$225
Consumables Expenses: \$2
Your profit: \$223
Warranty: 3-Year

OTHER BRANDS

Price per Treatment Average: \$175
Consumables Expenses: Average: \$55
Your profit: \$120
Warranty: 1-Year

Glass Skin Serums Collection

EXPERIENCE THE LUXURY of Zemits HydroDermabrasion Serums

Hydro-Exfoliation and Hydro-Extraction Serums

The Zemits Hydro-Exfoliation Serum Series is designed to provide effective exfoliation and purification for a clearer, smoother complexion.

Paired with the HydroDiamond Tips, these serums target specific skin concerns, offering a customized exfoliation experience tailored to each client's needs.



Serum Options:



Zemits HydroClear Pro:

Deeply clarifies and exfoliates, smoothing skin texture and reducing blemishes.



Zemits SalicPure Pro:

Purifies and unclogs pores, reducing acne and calming inflammation.



Zemits GlycoVelvet Pro:

Brightens and renews, targeting fine lines and uneven skin tone for a radiant, youthful glow.

Deep Hydration Infusion Serums

The Zemits Infusion Serum Series is designed for deep hydration and rejuvenation.

Applied with the Oxygen Infusion Gun, these serums deliver a potent flow of nutrients that penetrate deeply into the skin.

This infusion treatment revitalizes and hydrates, addressing specific skin needs to achieve a smooth, balanced, and radiant complexion.



Serum Options:



Zemits RevitaOx Pro

Deep hydration and antioxidant protection for plump, calm skin.



Zemits LuminOxy Pro

Brightens, hydrates, and smooths for a radiant complexion.



Zemits OxyTight Pro

Firms and boosts collagen to reduce fine lines and enhance elasticity.

Collagen-Boosting Conductive Gel

This collagen-boosting conductive gel is designed to deliver deep hydration, support collagen production, and provide a calming effect during aesthetic treatments.

Versatile and suitable for all skin types, it enhances treatment outcomes by nourishing the skin and improving the absorption of active ingredients. Ideal for use with Cold Therapy and other rejuvenating treatments.



Zemits FirmingGenius Pro:

Delivers intense hydration to keep skin soft and plump, stimulates collagen for firmer, youthful skin, and calms inflammation, resulting in a balanced and revitalized complexion.



Zemits EquiSheer Pro:

Essential Cleansing and Sanitizing for HydroDermabrasion Machines

What is Zemits EquiSheer Pro?

Zemits EquiSheer Pro is a specialized cleansing and sanitizing solution designed specifically for HydroDermabrasion machines. This solution is critical for maintaining your equipment's performance, preventing product buildup, and ensuring a hygienic environment for every treatment.



KEY BENEFITS:

Effective Cleaning:

Removes product buildup, ensuring your equipment stays in top condition.

Sanitizes Equipment:

Disinfects thoroughly, maintaining a hygienic treatment environment.

Extends Lifespan:

Protects surfaces, preventing wear and prolonging equipment life.

Maintains Performance:

Keeps your HydroDermabrasion system performing at its best.

Enhanced Disinfection:

Boosts sanitizing effectiveness for thorough protection.

USAGE INSTRUCTIONS:

- 01** | Fill the jar with 5 ml of Zemits EquiSheer Pro
- 02** | Attach a wand tip to the machine
- 03** | Create vacuum suction by placing your thumb on the tip while wearing gloves
- 04** | Start the machine and run the entire solution through it
- 05** | Finish the process and see the collected debris in the jar, showing just how effectively your machine has cleansed the skin during the treatment.



Zemits HydroDiamond™ Technology Tip Set

The Zemits HydroDiamond™ Technology comes with a versatile set of tips, each designed for specific skin concerns and treatment goals.

1. DUAL DIAMOND TIPS:

Unique Design:

Unlike traditional tips that only feature diamonds on the outer ring, these tips are fully covered with diamonds across the entire surface. This ensures a more comprehensive and effective exfoliation, removing dead skin cells and revealing smoother skin.

Smaller Opening:

The smaller opening in these tips reduces the risk of overstretching the skin during treatment, providing a gentler yet effective exfoliation experience.



H200 grit:

For precise skin hydro-exfoliation for normal to oily skin

H150 grit:

For moderate exfoliation for the face and neck for normal to oily skin

H100 grit:

For effective body hydro-exfoliation

2. RUFFLE PEEL TIPS:



Large Clear Ruffle Tip

This tip is designed specifically for clients with sensitive or reactive skin.

It has a textured surface that is less aggressive than diamond tips, providing a gentle resurfacing treatment.



Small Clear Ruffle Tip

This tip is perfect for detailed work around delicate areas like the nose and mouth.

It allows for precise treatment without causing irritation.

3. HYDRO EXTRACTION TIPS:

These tips are specialized for extractions, helping to remove sebum, blackheads, and other impurities from the skin.



Large Hydro Extraction Tip

For pain-free deep extractions of sebum and blackheads

Suitable for broader areas of the face, allowing for efficient extraction across larger surfaces



Small Hydro Extraction Tip

For precise non-traumatic extractions

Ideal for precise extraction around the nose and mouth where pores are often more congested.

HydroDiamond Timeless Glow

Anti-Aging Treatment Protocol

Products to Use:

- Zemits GlycoVelvet Pro Advanced Anti-Aging HydroDermabrasion Serum with Glycolic Acid & Vitamin C
- Zemits OxyTight Pro Repairing & Boosting Infusion Serum with Peptides
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides

Series of Treatments:

6-8 treatments recommended



ZEMITS GLYCOVELVET PRO:

- Exfoliates dead skin
- Smooths skin texture
- Reduces fine lines
- Boosts collagen
- Brightens and evens tone



ZEMITS OXYTIGHT PRO

- Stimulates collagen
- Firms and tightens skin
- Reduces fine lines
- Deeply hydrates and plumps
- Calms and soothes inflammation



ZEMITS FIRMINGGENIUS PRO

- Boosts collagen
- Tightens and firms skin
- Reduces wrinkles
- Enhances elasticity
- Deeply nourishes skin

STEPS

1. Double Cleansing:

- Begin with double cleansing the skin on the face, neck, and décolleté to remove impurities and prepare the skin for treatment.

2. Exfoliation and Anti-Aging:

- Apply 2-3 ml of Zemits GlycoVelvet Pro serum.
- Zemits GlycoVelvet Pro uses Glycolic Acid to exfoliate dead skin cells and Vitamin C to brighten the skin, reduce fine lines, and boost collagen production, enhancing overall skin texture and tone.

3. HydroDermabrasion:

- Perform the HydroDiamond step using the Dual Diamond Hydro Tip. This step deeply exfoliates the skin, removing dead skin cells and enhancing the penetration of active ingredients.
- For delicate areas around the eyes, switch to the Clear Ruffle tip. This tip is specifically designed for sensitive skin, providing gentle yet effective exfoliation around the eyes.

4. Hydro Extractions:

- After the HydroDermabrasion, perform Hydro Extractions using a clear smooth hydro tip. This step is essential for removing any remaining impurities and excess sebum from the pores, ensuring a thoroughly cleansed and clarified complexion.

5. Hydration and Repair:

- Apply Zemits OxyTight Pro serum, rich in Palmitoyl Tripeptide-5 and Hyaluronic Acid, to stimulate collagen production and deeply hydrate the skin.

6. Cold Skin Tightening:

- Topically apply Zemits FirmingGenius Pro gel and use slow circular motions with the DermeLuxx Cold Wand.
- The peptides in Zemits FirmingGenius Pro help stimulate collagen production, tighten the skin, and reduce the appearance of wrinkles. Focus on areas with visible signs of aging, using the Cold Wand to further enhance skin firmness.

7. Sunscreen Application:

- Complete the treatment with a sunscreen application to protect the skin from UV damage.



HydroDiamond VitaFusion


Super Hydration Treatment Protocol

Products to Use:

- Zemits HydroClear Pro Clarifying HydroDermabrasion Serum with Lactic Acid
- Zemits RevitaOx Pro Hydro-Revival Antioxidant Infusion Serum with Hyaluronic Acid
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides

Series of Treatments:

4-6 treatments recommended

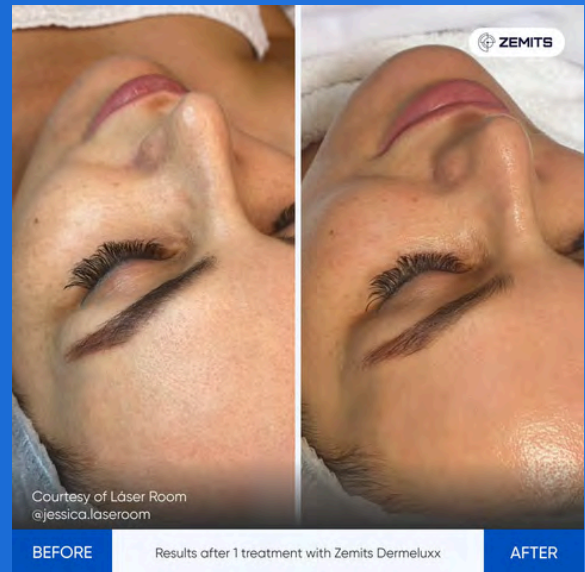

ZEMITS HYDROCLEAR PRO

- Deep exfoliation
- Enhances absorption
- Clarifies skin texture
- Reduces fine lines
- Prepares skin for hydration



ZEMITS REVITAOX PRO

- Intensive hydration
- Antioxidant protection
- Soothes irritation
- Improves elasticity
- Revitalizes skin



ZEMITS FIRMINGGENIUS PRO

- Boosts collagen
- Tightens and firms skin
- Reduces wrinkles
- Enhances elasticity
- Deeply nourishes skin

STEPS

1. Double Cleansing:

- Begin with double cleansing the skin on the face, neck, and décolleté to remove impurities and prepare the skin for optimal hydration.

2. Exfoliation and Clarification:

- Apply 2-3 ml of Zemits HydroClear Pro serum all over the treated area.
- Zemits HydroClear Pro, with its Lactic Acid and Kelp extracts, provides gentle exfoliation that preps the skin by removing dead cells, allowing better absorption of hydrating serums in the following steps.

3. HydroDermabrasion:

- Perform the HydroDiamond step using the Dual Diamond Hydro Tip to exfoliate the skin thoroughly, enhancing its ability to absorb moisture.
- Switch to the Clear Ruffle tip for the delicate areas around the eyes to ensure gentle exfoliation without irritation.

4. Hydro Extractions:

- After HydroDermabrasion, perform Hydro Extractions with a clear smooth hydro tip. This step helps to remove impurities and excess oil, leaving the skin clean and ready for intense hydration.

5. Hydration and Antioxidant Infusion:

- Perform an oxygen infusion with Zemits RevitaOx Pro serum.
- Zemits RevitaOx Pro contains Hyaluronic Acid and antioxidants, which deeply hydrate the skin, locking in moisture while protecting it from environmental damage. The result is plump, supple, and radiant skin.

6. Cold Skin Tightening:

- Apply Zemits FirmingGenius Pro gel and use the DermeLuxx Cold Wand in slow circular motions.
- The peptides and hydrating agents in Zemits FirmingGenius Pro gel help to lock in moisture, firm the skin, and enhance the overall hydration effect of the treatment.

7. Sunscreen Application:

- Finish the treatment with sunscreen to protect the newly hydrated skin from UV exposure.

HydroDiamond SkinDetox

Anti-Acne Treatment Protocol

Products to Use:

- Zemits SalicPure Pro Purifying HydroDermabrasion Serum with Salicylic Acid
- Zemits RevitaOx Pro Hydro-Revival Antioxidant Infusion Serum with Hyaluronic Acid
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides

Series of Treatments:

6-8 treatments recommended



ZEMITS SALICPURE PRO

- Purifies and exfoliates
- Unclogs pores
- Reduces acne
- Calms inflammation
- Improves clarity



ZEMITS REVITAOX PRO

- Hydrates skin
- Provides antioxidant protection
- Soothes irritation
- Aids healing
- Clarifies complexion



ZEMITS FIRMINGGENIUS PRO

- Boosts collagen
- Firms skin
- Reduces scars
- Improves elasticity
- Nourishes skin

STEPS

1. Double Cleansing:

- Start with double cleansing the skin on the face, neck, and décolleté to remove impurities, excess oil, and makeup, preparing the skin for treatment.

2. Exfoliation and Purification:

- Apply 2-3 ml of Zemits SalicPure Pro serum all over the treated area.
- Zemits SalicPure Pro is formulated with Salicylic Acid, which exfoliates and purifies the skin, unclogging pores, and reducing acne. Aloe Vera and Turmeric Root Extracts calm and soothe the skin, reducing redness and irritation.

3. HydroDermabrasion:

- Perform the HydroDiamond step using the Dual Diamond Hydro Tip to deeply exfoliate the skin and remove dead skin cells, reducing the likelihood of clogged pores.
- Use the Clear Ruffle tip for the delicate areas around the eyes, ensuring gentle yet effective exfoliation.

4. Hydro Extractions:

- After HydroDermabrasion, perform Hydro Extractions using a clear smooth hydro tip to remove remaining impurities, excess sebum, and debris from the pores, ensuring a thoroughly cleansed and clarified complexion.

5. Hydration and Antioxidant Infusion:

- Apply Zemits RevitaOx Pro serum, which deeply hydrates the skin while providing antioxidant protection. This serum helps to calm inflammation, reduce acne-related irritation, and promote overall skin health with Hyaluronic Acid and Grape Seed Extract.

6. Cold Skin Tightening:

- Apply Zemits FirmingGenius Pro gel and use the Cold Wand in slow circular motions. This step helps to soothe the skin, reduce post-acne scarring, and enhance skin firmness.

7. Sunscreen Application:

- Complete the treatment with sunscreen to protect the skin from UV damage and prevent further irritation or post-acne hyperpigmentation.

HydroDiamond Luminance

Skin Brightening Treatment Protocol

Products to Use:

- Zemits SalicPure Pro Purifying HydroDermabrasion Serum with Salicylic Acid
- Zemits LuminOxy Pro Brighten & Recharge Hydrating Infusion Serum with Glycolic Acid
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides

Series of Treatments:

4-6 treatments recommended



ZEMITS SALICPURE PRO

- Exfoliates and purifies
- Unclogs pores, reduces acne
- Brightens and evens tone
- Calms inflammation
- Improves skin clarity



ZEMITS LUMINOXY PRO

- Brightens and evens tone
- Reduces dark spots
- Hydrates and plumps
- Stimulates collagen
- Enhances radiance



ZEMITS FIRMINGGENIUS PRO

- Boosts collagen
- Firms and tightens
- Reduces fine lines
- Enhances elasticity
- Deeply nourishes

STEPS

1. Double Cleansing:

- Start with double cleansing the skin on the face, neck, and décolleté to remove impurities, makeup, and excess oil, preparing the skin for treatment.

2. Exfoliation and Brightening:

- Apply 2-3 ml of Zemits SalicPure Pro serum all over the treated area.
- Zemits SalicPure Pro uses Salicylic Acid to exfoliate the skin and Turmeric Root Extract to brighten and even out skin tone, helping reduce dark spots and pigmentation.

3. HydroDermabrasion:

- Perform the HydroDiamond step using the Dual Diamond Hydro Tip to deeply exfoliate the skin, removing dead skin cells and enhancing the absorption of brightening serums.
- Use the Clear Ruffle tip for gentle treatment around the eyes to brighten the delicate skin in this area.

4. Hydro Extractions:

- Perform Hydro Extractions using a clear smooth hydro tip to remove impurities and excess sebum from the pores, ensuring a clean and clarified complexion.

5. Brightening and Hydration Infusion:

- Apply Zemits LuminOxy Pro serum, which combines Glycolic Acid with brightening agents like Bearberry Leaf Extract and Lemon Peel Oil. This serum helps to even out skin tone, reduce hyperpigmentation, and hydrate the skin for a radiant glow.

6. Cold Skin Tightening:

- Apply Zemits FirmingGenius Pro gel and use the Cold Wand in slow circular motions. This step helps to firm the skin, enhance elasticity, and lock in moisture, further enhancing the brightening effect.

7. Sunscreen Application:

- Complete the treatment with sunscreen to protect the skin from UV damage and prevent further pigmentation.



HydroDiamond NeckTite

Neck Rejuvenation Treatment Protocol

Products to Use:

- Zemits HydroClear Pro Clarifying HydroDermabrasion Serum with Lactic Acid
- Zemits OxyTight Pro Repairing & Boosting Infusion Serum with Peptides
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides



Series of Treatments:

'6-8 treatments recommended



ZEMITS HYDROCLEAR PRO

- Deeply exfoliates and clarifies
- Smooths texture
- Reduces fine lines
- Prepares skin for serum absorption
- Promotes firmer, youthful skin



ZEMITS OXYTIGHT PRO

- Stimulates collagen
- Firms and tightens skin
- Reduces fine lines
- Deeply hydrates and plumps
- Calms inflammation



ZEMITS FIRMINGGENIUS PRO

- Boosts collagen
- Firms and tightens skin
- Reduces fine lines
- Enhances elasticity
- Provides deep nourishment

STEPS

1. Double Cleansing:

- Start with double cleansing the skin on the neck and jawline to remove impurities, oil, and makeup, preparing the skin for treatment.

2. Exfoliation and Clarification:

- Apply 2-3 ml of Zemits HydroClear Pro serum all over the neck and jawline area.
- Zemits HydroClear Pro uses Lactic Acid to gently exfoliate the skin, removing dead skin cells and promoting smoother, firmer skin, which is essential for rejuvenating the neck area.

3. HydroDermabrasion:

- Perform the HydroDiamond step using the Dual Diamond Hydro Tip to deeply exfoliate and smooth the skin on the neck and jawline, improving texture and preparing the skin for further treatment.

4. Hydro Extractions:

- Perform Hydro Extractions using a clear smooth hydro tip to remove any remaining impurities and excess sebum from the pores, ensuring a clean and clarified complexion.

5. Hydration and Firming Infusion:

- Apply Zemits OxyTight Pro serum, which contains peptides and Hyaluronic Acid to firm and tighten the skin, reducing the appearance of fine lines and wrinkles on the neck. This serum also enhances collagen production, improving skin elasticity.

6. Cold Skin Tightening:

- Apply Zemits FirmingGenius Pro gel and use the Cold Wand in slow circular motions on the neck and jawline area. This step helps to further tighten the skin, reduce sagging, and enhance the contour of the neck.

7. Sunscreen Application:

- Complete the treatment with sunscreen to protect the neck area from UV damage and maintain the results.



HydroDiamond ScalpBoost

Scalp Treatment Protocol

Products to Use:

- Zemits SalicPure Pro Purifying HydroDermabrasion Serum with Salicylic Acid
- Zemits RevitaCollagen Collagen-Boosting Meso-Therapy Serums
- Zemits PureKio 2.0 Professional High-Frequency System

Series of Treatments:

4-6 sessions recommended



ZEMITS SALICPURE PRO

- Purifies the scalp
- Unclogs hair follicles
- Calms inflammation
- Balances oil production
- Enhances serum absorption



ZEMITS REVITACOLLAGEN

- Nourishes hair follicles
- Stimulates hair growth
- Strengthens hair roots
- Increases hair density
- Promotes healthier scalp



ZEMITS PUREKIO 2.0

- Boosts scalp circulation
- Stimulates hair follicles
- Promotes scalp oxygenation
- Reduces dandruff and inflammation
- Enhances serum absorption

STEPS

1. Pre-Treatment Preparation:

- Ensure the treatment area is clean and organized. Conduct a detailed consultation with the client to understand their scalp condition, concerns, and desired outcomes.

2. Scalp Preparation and Exfoliation:

- Brushing and Parting Hair: Begin by gently brushing the client's hair to remove tangles, and part the hair to provide access to the scalp.
- Application of Zemits SalicPure Pro: Apply 2-3 ml of Zemits SalicPure Pro serum evenly across the scalp using a fan brush. This serum purifies and exfoliates the scalp, unclogging hair follicles and preparing the scalp for enhanced absorption of the hair growth serum.
- Exfoliation with Ruffle Tip: Use the HydroDiamond device with the ruffle tip attachment to exfoliate the scalp. This step removes dead skin cells, promotes blood circulation, and prepares the scalp for better absorption of treatment serums.

3. Serum Infusion:

- Apply Zemits RevitaCollagen using a suitable device or manually. This step is crucial as it deeply nourishes and stimulates hair follicles, promoting healthy and robust hair growth. Zemits RevitaCollagen strengthens hair follicles, enhances scalp health by replenishing collagen, and improves elasticity and hydration, creating an optimal environment for healthy hair growth.

4. Optional Step: Microneedling:

- Microneedling (Optional): If microneedling is provided, this step can be added after the serum infusion and before the high-frequency treatment. Microneedling enhances the absorption of the hair growth serum and stimulates the scalp for better results.

5. High-Frequency Application:

- Preparation: Prepare the Zemits PureKio 2.0 Professional High-Frequency System and attach the comb electrode, setting the intensity level according to the client's comfort.
- Application: Gently glide the comb electrode across the scalp in slow, deliberate motions. This high-frequency treatment increases blood circulation, stimulates hair follicles, promotes oxygenation of the scalp, and reduces dandruff and inflammation.

6. Post-Treatment Care:

- Advise the client on post-treatment care, including any specific products or routines to maintain scalp health and maximize treatment results.
- Schedule follow-up appointments as needed to monitor progress and adjust the treatment plan accordingly.

FREQUENCY OF TREATMENT:

- For general scalp maintenance, treatments can be performed every 4-6 weeks.
- For more specific concerns like hair thinning, treatments may be recommended every 2-3 weeks until desired results are achieved.

NUMBER OF SESSIONS:

- Noticeable improvements are typically seen after 4-6 sessions. However, for more significant concerns, additional sessions may be necessary, based on the client's individual needs.



WATCH THE SCALP TRANSFORMATION:

Step-by-Step Video Guide



WATCH NOW 

HydroDiamond SilkHands

Hand Rejuvenation Treatment Protocol

Products to Use:

- Zemits HydroClear Pro Clarifying HydroDermabrasion Serum with Lactic Acid
- Zemits RevitaOx Pro Hydro-Revival Antioxidant Infusion Serum with Hyaluronic Acid
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides

Series of Treatments:

4 sessions recommended, once a month



ZEMITS HYDROCLEAR PRO

- Deep exfoliation
- Smooths skin texture
- Enhances serum absorption
- Brightens the skin
- Promotes cell renewal



ZEMITS REVITAOX PRO

- Deeply hydrates skin
- Boosts collagen
- Improves elasticity
- Brightens and evens tone
- Reduces fine lines and wrinkles



ZEMITS FIRMINGGENIUS PRO

- Boosts collagen
- Firms and tightens skin
- Reduces fine lines and wrinkles
- Enhances elasticity
- Deeply nourishes and hydrates

STEPS

1. Pre-Treatment Preparation:

- Ensure the treatment area is clean, sanitized, and equipped with all necessary tools and products for the hand rejuvenation treatment.
- Conduct a thorough consultation with the client to assess their concerns, skin type, and goals for the treatment.

2. HydroDiamond Exfoliation:

- HydroDiamond Wand Preparation: Prepare the HydroDiamond wand, ensuring it is properly sanitized and equipped with the dual diamond hydro tips attachment.
- Application of Zemits HydroClear Pro: Apply 2-3 ml of Zemits HydroClear Pro serum evenly across the hands. This serum, formulated with Lactic Acid, gently exfoliates the skin, promoting cell turnover and revealing smoother, more radiant hands.
- Dual Diamond Hydro Tips Exfoliation: Use the HydroDiamond wand in gentle circular motions across the hands, targeting areas of roughness or uneven texture. This step effectively removes dead skin cells and prepares the hands for better absorption of subsequent serums.

3. Oxygen Serum Infusion:

- Serum Infusion Preparation: Prepare the infusion gun and load it with Zemits RevitaOx Pro serum, formulated with Hyaluronic Acid and antioxidants for deep hydration and skin renewal.
- Oxygen Serum Infusion: Use the infusion gun to apply Zemits RevitaOx Pro serum evenly across the hands. The gentle pressure and oxygenation process drive the serum deep into the skin, delivering potent antioxidants and moisture-binding Hyaluronic Acid.

4. Cold Therapy Application:

- Peptide Gel Application: Apply a generous layer of Zemits FirmingGenius Pro gel to the hands, covering the entire treated area. This peptide-rich gel soothes and rejuvenates the skin while providing a cooling sensation.
- Cold Therapy Application: Utilize the Cold Wand to gently massage the peptide gel into the skin. Cold therapy helps to reduce inflammation, soothe irritation, and improve circulation, leaving the hands feeling refreshed and revitalized.



5. OxiVelour Enhancement:

- Oxygen Therapy During Hand Treatment: While the hand treatment is in progress, position the OxyBubble dome over the client's face. This dome delivers oxygen-enriched therapy, enhancing skin clarity, hydration, and overall radiance. The client can enjoy a relaxing facial treatment while their hands are being rejuvenated.
- Comprehensive Rejuvenation: The OxyBubble dome not only enhances the facial treatment but also adds to the overall relaxation and rejuvenation experience, making it a perfect complementary treatment during the hand rejuvenation session

6. Neck Rejuvenation with Zemits EcliPure:

- Simultaneous Neck Treatment: As an additional enhancement, apply the Zemits EcliPure Light System for neck rejuvenation during the hand treatment. This light therapy targets fine lines and sagging skin on the neck, providing firming and tightening benefits.

7. Post-Treatment Care:

- Advise the client on post-treatment care, including the use of sunscreen and moisturizers to protect and maintain the results of the hand rejuvenation treatment.
- Recommend follow-up appointments as needed to monitor progress and adjust the treatment plan based on the client's individual response.

ZEMITS ECLIPURE

- Firms and tightens neck skin
- Boosts collagen and elasticity
- Reduces redness and inflammation
- Non-invasive anti-aging
- Targets delicate neck area




ZEMITS OXIVELOUR

- Deeply hydrates skin
- Boosts radiance and clarity
- Enhances relaxation
- Improves oxygenation
- Complements other treatments

WATCH THE HANDS TRANSFORMATION:

Step-by-Step Video Guide

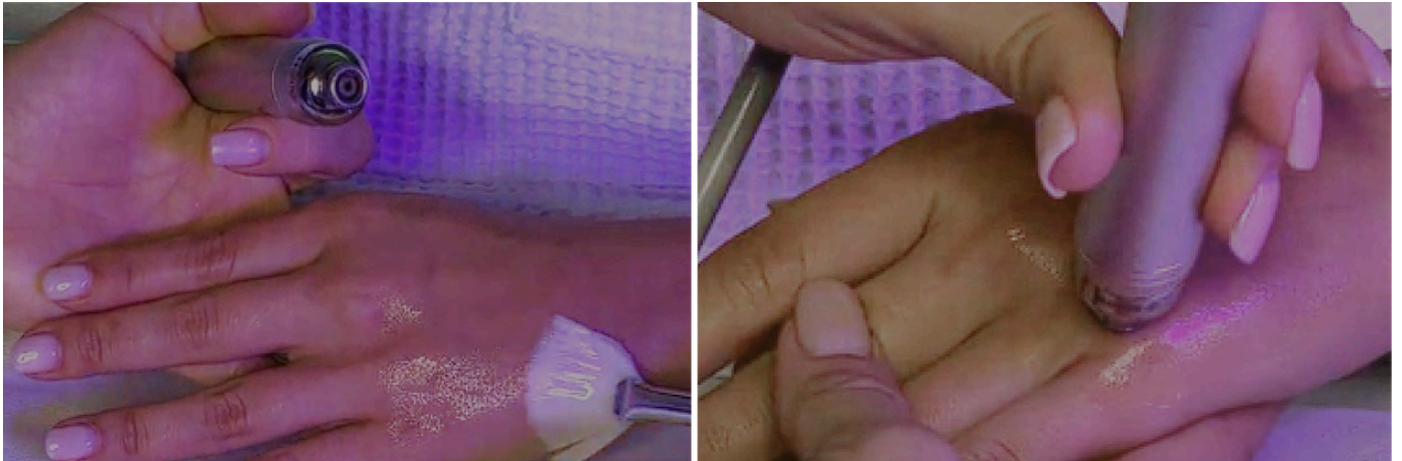

WATCH NOW 

HydroDiamond BackPurify

Back Facial Treatment Protocol

Products to Use:

- Zemits SalicPure Pro Purifying HydroDermabrasion Serum with Salicylic Acid
- Zemits LuminOxy Pro Brighten & Recharge Hydrating Infusion Serum with Glycolic Acid
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides

Series of Treatments:

Every 4-6 weeks recommended



ZEMITS SALICPURE PRO

- Unclogs pores and reduces acne
- Exfoliates and purifies
- Soothes inflammation
- Improves skin clarity
- Balances oil production



ZEMITS LUMINOXY PRO

- Brightens skin tone
- Reduces dark spots
- Hydrates and revitalizes
- Promotes radiance
- Enhances skin clarity



ZEMITS FIRMINGGENIUS PRO

- Boosts collagen
- Firms and tightens skin
- Reduces fine lines and wrinkles
- Enhances elasticity
- Deeply nourishes and hydrates

STEPS

1. Pre-Treatment Preparation:

- Ensure the treatment area is clean, sanitized, and equipped with all necessary tools and products.
- Conduct a thorough consultation with the client to assess their skin concerns, sensitivities, and goals for the treatment.

2. Cleansing and Exfoliation:

- Cleansing: Begin by thoroughly cleansing the back to remove any surface impurities and prepare the skin for treatment.
- Application of Zemits SalicPure Pro: Apply 2-3 ml of Zemits SalicPure Pro serum evenly across the back. This serum, formulated with Salicylic Acid, gently exfoliates the skin, unclogs pores, and helps to reduce acne and congestion.
- Exfoliation with Dual Hydro Tip: Utilize the HydroDiamond wand with the dual hydro tip attachment to perform gentle exfoliation. Move the wand in circular motions across the back, targeting areas of roughness and uneven texture. This step sloughs off dead skin cells and promotes smoother, healthier-looking skin.

3. Extraction:

- Extraction with Clear Hydro Tip: Switch to the clear hydro tip attachment on the HydroDiamond wand to perform gentle suction-based extraction. Focus on congested areas, targeting blackheads and whiteheads for removal. This step helps to decongest pores and improve skin clarity.

4. Oxygen Serum Infusion:

- Serum Infusion Preparation: Prepare the infusion gun and load it with Zemits LuminOxy Pro serum, formulated with Glycolic Acid and brightening agents like Bearberry Leaf Extract and Lemon Peel Oil.
- Oxygen Serum Infusion: Use the infusion gun to apply Zemits LuminOxy Pro serum evenly across the back. The gentle pressure and oxygenation process deliver the brightening ingredients deep into the skin, promoting a more even skin tone and radiant complexion.



5. Cold Therapy Application:

- **Peptide Gel Application:** Apply a generous layer of Zemits FirmingGenius Pro gel to the back, covering the entire treated area. This peptide-rich gel soothes and rejuvenates the skin while providing a cooling sensation.
- **Cold Therapy Application:** Utilize cold therapy techniques, such as chilled massage balls or cold compresses, to gently massage the peptide gel into the skin. Cold therapy helps to reduce inflammation, soothe irritation, and improve circulation, leaving the back feeling refreshed and revitalized.

6. Post-Treatment Care:

- Advise the client on post-treatment care, including the use of sunscreen and moisturizers to protect and maintain the results of the back facial treatment.
- Recommend follow-up appointments as needed to monitor progress and adjust the treatment plan based on the client's individual response.



HydroDiamond Velvety

Pores Refinement Treatment Protocol

Products to Use:

- Zemits SalicPure Pro Purifying HydroDermabrasion Serum with Salicylic Acid
- Zemits OxyTight Pro Repairing & Boosting Infusion Serum with Peptides
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides

Series of Treatments:

4-6 treatments recommended



ZEMITS SALICPURE PRO

- Unclogs pores
- Reduces acne
- Exfoliates and purifies
- Soothes inflammation
- Improves skin clarity



ZEMITS OXYTIGHT PRO

- Tightens pores
- Boosts collagen
- Firms and smooths skin
- Enhances elasticity
- Calms irritation



ZEMITS FIRMINGGENIUS PRO

- BoBoosts collagen
- Firms and tightens skin
- Reduces fine lines
- Enhances elasticity
- Nourishes and hydrates

STEPS

1. Pre-Treatment Preparation:

- Ensure the treatment area is clean and equipped with all necessary tools and products.
- Conduct a consultation with the client to assess their skin concerns, focusing on pore size, oil production, and skin texture.

2. Cleansing and Exfoliation:

- Cleansing: Start by thoroughly cleansing the face to remove any surface impurities, makeup, and excess oil.
- Application of Zemits SalicPure Pro: Apply 2-3 ml of Zemits SalicPure Pro serum. This serum exfoliates the skin, unclogs pores, and purifies the complexion, helping to reduce pore size and prevent congestion.
- Exfoliation with HydroDiamond: Use the HydroDiamond wand with a suitable tip to gently exfoliate the skin. Focus on areas with visible pores and excess oil production. This step helps to refine the skin texture and promote a smoother, clearer complexion.

3. Oxygen Serum Infusion:

- Application of Zemits OxyTight Pro: Apply Zemits OxyTight Pro serum to infuse the skin with peptides that firm and tighten the pores while promoting collagen production. This serum helps to enhance skin elasticity and provide a smooth finish.

4. Cold Therapy:

- Application of Zemits FirmingGenius Pro: Apply Zemits FirmingGenius Pro gel and use a Cold Wand to gently massage the skin. Cold therapy helps to further tighten pores, reduce redness, and soothe the skin, leaving it feeling refreshed.

5. Post-Treatment Care:

- Advise the client on post-treatment care, including the use of a gentle cleanser, oil-free moisturizer, and sunscreen to protect the skin and maintain the refined pore appearance.
- Recommend a follow-up schedule of 4-6 treatments for optimal pore refinement results.



HydroDiamond SolarCare

Post-Sun Care Treatment Protocol

Products to Use:

- Zemits RevitaOx Pro Hydro-Revival Antioxidant Infusion Serum with Hyaluronic Acid
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides

Series of Treatments:

4-6 treatments recommended



ZEMITS REVITAOX PRO

- Deeply hydrates
- Provides antioxidant protection
- Soothes and calms skin
- Reduces redness
- Aids skin recovery



ZEMITS FIRMINGGENIUS PRO

- Cools and soothes
- Boosts collagen
- Firms and tightens skin
- Reduces fine lines
- Nourishes and hydrates

STEPS

1. Pre-Treatment Preparation:

- Ensure the treatment area is clean and prepared with all necessary tools and products.
- Conduct a consultation with the client to assess their skin's condition after sun exposure, focusing on redness, dehydration, and any signs of damage.

2. Gentle Cleansing:

- Cleansing: Start by gently cleansing the skin to remove any surface impurities, sunscreen, and excess oil. Avoid harsh cleansers that could further irritate sun-exposed skin.

3. Hydration and Soothing Serum Infusion:

- Application of Zemits RevitaOx Pro: Apply Zemits RevitaOx Pro serum to infuse the skin with antioxidants and Hyaluronic Acid. This serum hydrates, repairs, and protects the skin from further damage while calming inflammation and soothing irritation.

4. Cold Therapy:

- Application of Zemits FirmingGenius Pro: Apply Zemits FirmingGenius Pro gel and use a Cold Wand to gently massage the skin. Cold therapy helps to soothe sun-damaged skin, reduce redness, and lock in hydration, leaving the skin refreshed and revitalized.

5. Post-Treatment Care:

- Advise the client on post-treatment care, including the use of a gentle moisturizer and sunscreen to protect the skin and prevent further sun damage.
- Recommend a follow-up schedule of 4-6 treatments to fully repair and restore the skin after sun exposure.



HydroDiamond EvenTone

Hyperpigmentation Treatment Protocol

Products to Use:

- Zemits GlycoVelvet Pro Advanced Anti-Aging HydroDermabrasion Serum with Glycolic Acid
- Zemits LuminOxy Pro Brighten & Recharge Hydrating Infusion Serum with Glycolic Acid
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides

Series of Treatments

4-6 treatments recommended



ZEMITS GLYCOVELVET PRO

- Exfoliates and smooths skin
- Reduces dark spots
- Promotes cell turnover
- Evens skin tone
- Brightens complexion



ZEMITS LUMINOXY PRO

- Brightens and evens skin tone
- Reduces hyperpigmentation
- Hydrates skin
- Enhances radiance
- Improves clarity



ZEMITS FIRMINGGENIUS PRO

- Firms and tightens skin
- Boosts collagen
- Reduces fine lines
- Soothes and hydrates
- Enhances elasticity

STEPS

1. Pre-Treatment Preparation:

- Ensure the treatment area is clean and equipped with all necessary tools and products.
- Conduct a consultation with the client to assess their skin concerns, focusing on hyperpigmentation, sunspots, and uneven skin tone.

2. Cleansing and Exfoliation:

- Cleansing: Gently cleanse the skin to remove impurities, makeup, and excess oil, preparing it for the treatment.
- Application of Zemits GlycoVelvet Pro: Apply 2–3 ml of Zemits GlycoVelvet Pro serum. This serum exfoliates the skin using Glycolic Acid, promoting cell turnover, and reducing the appearance of dark spots and uneven skin tone.
- Exfoliation with HydroDiamond: Use the HydroDiamond wand with a suitable tip to perform a gentle exfoliation, focusing on areas with visible hyperpigmentation. This step helps to smooth the skin and prepare it for the infusion of brightening serums.

3. Oxygen Serum Infusion:

- Application of Zemits LuminOxy Pro: Apply Zemits LuminOxy Pro serum to infuse the skin with brightening agents that help to reduce hyperpigmentation and even out skin tone. The serum's combination of Glycolic Acid and other brightening ingredients works to enhance skin clarity and radiance.

4. Cold Therapy:

- Application of Zemits FirmingGenius Pro: Apply Zemits FirmingGenius Pro gel and use a Cold Wand to gently massage the skin. Cold therapy helps to soothe the skin, reduce redness, and lock in hydration, leaving the skin refreshed and revitalized.

5. Post-Treatment Care:

- Advise the client on post-treatment care, including the use of sunscreen to protect the skin from UV damage and prevent further hyperpigmentation.
- Recommend a follow-up schedule of 4–6 treatments to achieve optimal results in reducing hyperpigmentation.



HydroDiamond CalmBreeze

Rosacea Management Treatment Protocol

Products to Use:

- Zemits OxyTight Pro Repairing & Boosting Infusion Serum with Peptides
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides

Series of Treatments

4-6 treatments recommended



ZEMITS OXYTIGHT PRO BENEFITS:

- Soothes inflammation
- Reduces redness
- Strengthens skin barrier
- Hydrates and plumps skin
- Boosts collagen production



ZEMITS FIRMINGGENIUS PRO BENEFITS:

- Firms and tightens skin
- Reduces redness
- Calms and soothes
- Enhances skin elasticity
- Provides deep hydration

STEPS

1. Pre-Treatment Preparation:

- Ensure the treatment area is clean and equipped with all necessary tools and products.
- Conduct a thorough consultation with the client to assess the severity of their rosacea, skin sensitivities, and overall skin condition.

2. Gentle Cleansing:

- Cleansing: Start by gently cleansing the skin to remove any impurities and excess oil. Use a mild, non-irritating cleanser that is suitable for sensitive, rosacea-prone skin.

3. Oxygen Serum Infusion:

- Application of Zemits OxyTight Pro: Apply Zemits OxyTight Pro serum, which is rich in peptides and Hyaluronic Acid. This serum helps to repair and strengthen the skin barrier, soothe inflammation, and reduce redness, making it ideal for managing rosacea.

4. Cold Therapy:

- Application of Zemits FirmingGenius Pro: Apply Zemits FirmingGenius Pro gel and use a Cold Wand to gently massage the skin. Cold therapy is particularly beneficial for rosacea as it helps to calm inflamed skin, reduce redness, and provide a soothing, cooling effect.

5. Post-Treatment Care:

- Advise the client on post-treatment care, including the use of gentle, fragrance-free moisturizers and sunscreen to protect the skin from UV damage and environmental stressors that can exacerbate rosacea.
- Recommend a follow-up schedule of 4-6 treatments for optimal management of rosacea symptoms.



HydroDiamond GentleCare

Sensitive Skin Treatment Protocol

Products to Use:

- Zemits HydroClear Pro Clarifying HydroDermabrasion Serum with Lactic Acid
- Zemits RevitaOx Pro Hydro-Revival Antioxidant Infusion Serum with Hyaluronic Acid
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides



Series of Treatments:

4-6 treatments recommended



ZEMITS HYDROCLEAR PRO

- Gently exfoliates
- Reduces redness
- Calms and soothes skin
- Improves skin texture
- Prepares skin for hydration



ZEMITS REVITAOX PRO

- Deeply hydrates
- Soothes irritation
- Reduces inflammation
- Provides antioxidant protection
- Promotes skin healing



ZEMITS FIRMINGGENIUS PRO

- Firms and lifts skin
- Reduces redness
- Calms and soothes
- Locks in moisture
- Strengthens skin barrier

STEPS

1. Pre-Treatment Preparation:

- Ensure the treatment area is clean and prepared with all necessary tools and products.
- Conduct a thorough consultation with the client to assess their skin sensitivities and overall condition. Avoid any known irritants or allergens.

2. Gentle Cleansing and Exfoliation:

- Cleansing: Start with a gentle cleanse to remove any impurities, oil, and makeup. Use a mild cleanser suitable for sensitive skin.
- Application of Zemits HydroClear Pro: Apply 2-3 ml of Zemits HydroClear Pro serum. This serum gently exfoliates the skin using Lactic Acid, helping to clarify the complexion while maintaining the skin's moisture balance.
- Exfoliation with HydroDiamond: Perform a gentle exfoliation using the HydroDiamond wand. Focus on maintaining a light touch, especially on areas prone to sensitivity, to avoid irritation.

3. Oxygen Serum Infusion:

- Application of Zemits RevitaOx Pro: Apply Zemits RevitaOx Pro serum to infuse the skin with antioxidants and Hyaluronic Acid. This serum helps to soothe, hydrate, and protect the skin, making it ideal for sensitive skin types.

4. Cold Therapy:

- Application of Zemits FirmingGenius Pro: Apply Zemits FirmingGenius Pro gel and use a Cold Wand to gently massage the skin. Cold therapy helps to calm the skin, reduce redness, and lock in hydration, providing a soothing effect.

5. Post-Treatment Care:

- Advise the client on post-treatment care, emphasizing the use of gentle, fragrance-free moisturizers and sunscreen to protect the skin from environmental stressors.
- Recommend avoiding harsh treatments or products that could irritate the skin between sessions.



HydroDiamond Red Carpet

Pre-Event Glow Treatment Protocol

Products to Use:

- Zemits GlycoVelvet Pro Advanced Anti-Aging HydroDermabrasion Serum with Glycolic Acid
- Zemits LuminOxy Pro Brighten & Recharge Hydrating Infusion Serum with Glycolic Acid (Oxygen Serum Infusion)
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides

Series of Treatments:

1-2 days before the event



ZEMITS GLYCOVELVET PRO

- Smooths skin
- Brightens complexion
- Reduces dullness
- Preps skin for serums
- Reveals glow



ZEMITS LUMINOXY PRO

- Brightens skin
- Boosts radiance
- Deeply hydrates
- Evens tone
- Enhances clarity



ZEMITS FIRMINGGENIUS PRO

- Firms skin
- Reduces lines
- Locks in moisture
- Soothes skin
- Enhances firmness

STEPS

1. Pre-Treatment Preparation:

- Ensure the treatment area is clean and equipped with all necessary tools and products.
- Conduct a brief consultation with the client to understand their skin concerns and desired results for the event.

2. Cleansing and Exfoliation:

- Cleansing: Begin with a gentle cleanse to remove any surface impurities, makeup, and excess oil.
- Application of Zemits GlycoVelvet Pro: Apply 2–3 ml of Zemits GlycoVelvet Pro serum. This serum gently exfoliates the skin, promoting a smoother texture and brightening the complexion. It helps to remove dead skin cells and prepare the skin for the infusion of brightening serums.
- Exfoliation with HydroDiamond: Use the HydroDiamond wand with a suitable tip to perform a gentle exfoliation, focusing on areas where the skin may appear dull or rough. This step enhances skin radiance and smoothness.

3. Oxygen Serum Infusion:

- Application of Zemits LuminOxy Pro: Apply Zemits LuminOxy Pro serum to infuse the skin with brightening and hydrating ingredients. The combination of Glycolic Acid and other brightening agents helps to even out skin tone, enhance radiance, and give the skin a luminous glow.

4. Cold Therapy:

- Application of Zemits FirmingGenius Pro: Apply Zemits FirmingGenius Pro gel and use a Cold Wand to gently massage the skin. Cold therapy helps to tighten and firm the skin, reduce any puffiness, and lock in hydration, leaving the skin with a refreshed and glowing appearance.

5. Post-Treatment Care:

- Advise the client on post-treatment care, including the use of a gentle moisturizer and sunscreen if they will be exposed to sunlight before the event.
- Recommend minimal makeup application for the event to enhance the natural glow achieved from the treatment.

HydroDiamond GentleM


Men's Skin Care Treatment Protocol

Products to Use:

- Zemits HydroClear Pro Clarifying HydroDermabrasion Serum with Lactic Acid
- Zemits OxyTight Pro Repairing & Boosting Infusion Serum with Peptides
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides

Series of Treatments:

4-6 treatments recommended, depending on the client's skin needs

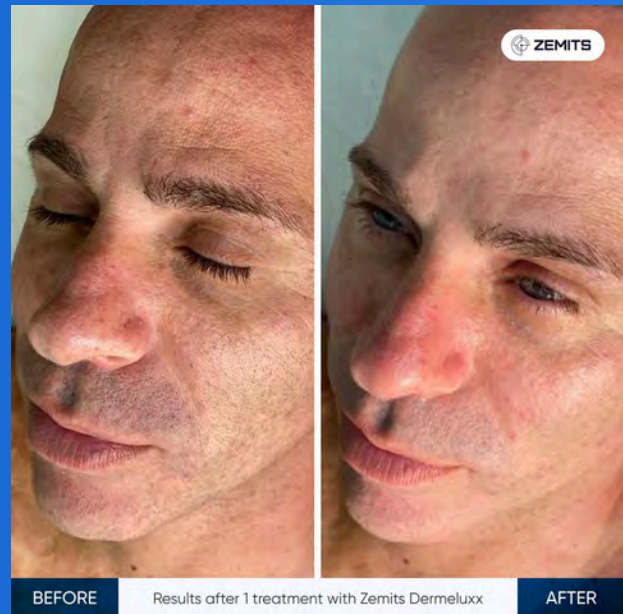

ZEMITS HYDROCLEAR PRO

- Deeply cleanses pores
- Smooths skin texture
- Reduces excess oil
- Clarifies complexion
- Prepares skin for hydration



ZEMITS OXYTIGHT PRO

- Firms and tightens
- Boosts elasticity
- Soothes redness
- Hydrates and repairs
- Enhances collagen production



ZEMITS FIRMINGGENIUS PRO

- Firms and lifts skin
- Reduces fine lines
- Soothes redness
- Locks in moisture
- Strengthens skin structure

STEPS

1. Pre-Treatment Preparation:

- Ensure the treatment area is clean and equipped with all necessary tools and products.
- Conduct a consultation with the client to understand their skin type, concerns, and goals. Focus on common issues in men's skincare such as oiliness, enlarged pores, rough texture, and signs of aging.

2. Cleansing and Exfoliation:

- Cleansing: Begin with a deep cleanse to remove impurities, excess oil, and dirt from the skin. Use a cleanser that suits the client's skin type, whether it's oily, dry, or combination.
- Application of Zemits HydroClear Pro: Apply 2-3 ml of Zemits HydroClear Pro serum. This serum uses Lactic Acid to gently exfoliate the skin, unclog pores, and clarify the complexion, addressing rough texture and uneven skin tone commonly found in men's skin.
- Exfoliation with HydroDiamond: Use the HydroDiamond wand with a suitable tip to perform a thorough exfoliation. Focus on areas with rough skin, enlarged pores, and any areas prone to ingrown hairs. This step helps to smooth the skin, reduce pore size, and prepare the skin for the infusion of serums.

3. Oxygen Serum Infusion:

- Application of Zemits OxyTight Pro: Apply Zemits OxyTight Pro serum to infuse the skin with peptides that help repair and strengthen the skin barrier. This serum also boosts collagen production, which is essential for maintaining skin firmness and reducing the appearance of fine lines, making it ideal for men's skincare.

4. Cold Therapy:

- Application of Zemits FirmingGenius Pro: Apply Zemits FirmingGenius Pro gel and use a Cold Wand to gently massage the skin. Cold therapy helps to soothe the skin, reduce any post-treatment redness, tighten pores, and enhance skin elasticity, leaving the skin feeling refreshed and revitalized.

5. Post-Treatment Care:

- Advise the client on post-treatment care, including the use of a suitable moisturizer and sunscreen to protect the skin from environmental stressors and UV damage.
- Recommend products that align with the client's skin type and concerns, such as oil-control products for oily skin or anti-aging serums for mature skin.

HydroDiamond ClearTeen

Teen Facial Treatment Protocol

Products to Use:

- Zemits SalicPure Pro Purifying HydroDermabrasion Serum with Salicylic Acid
- Zemits RevitaOx Pro Hydro-Revival Antioxidant Infusion Serum with Hyaluronic Acid
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides

Series of Treatments:

4-6 treatments recommended, spaced 2-4 weeks apart depending on the severity of skin concerns



ZEMITS SALICPURE PRO

- Unclogs pores
- Reduces acne
- Exfoliates gently
- Balances oil
- Improves clarity



ZEMITS REVITAOX PRO

- Hydrates skin
- Soothes irritation
- Provides antioxidants
- Promotes healing
- Enhances radiance



ZEMITS FIRMINGGENIUS PRO

- Firms and lifts skin
- Reduces redness
- Soothes and calms
- Boosts elasticity
- Locks in hydration

STEPS

1. Pre-Treatment Preparation:

- Ensure the treatment area is clean and equipped with all necessary tools and products.
- Conduct a consultation with the teen and, if necessary, their parent or guardian, to understand the specific skin concerns. Focus on common issues in teen skincare such as acne, oiliness, blackheads, and uneven skin tone.

2. Cleansing and Exfoliation:

- Cleansing: Begin with a gentle cleanse to remove impurities, excess oil, and any makeup. Use a cleanser that is suitable for young, acne-prone, or sensitive skin.
- Application of Zemits SalicPure Pro: Apply 2–3 ml of Zemits SalicPure Pro serum. This serum uses Salicylic Acid to exfoliate the skin, unclog pores, and reduce acne and blackheads. It also helps to purify the skin and manage oil production, which is essential for teenage skin prone to breakouts.
- Exfoliation with HydroDiamond: Use the HydroDiamond wand with a gentle tip to exfoliate the skin. Focus on areas with visible acne, blackheads, and rough texture. This step helps to smooth the skin, reduce pore size, and prepare the skin for the infusion of calming and hydrating serums.

3. Oxygen Serum Infusion:

- Application of Zemits RevitaOx Pro: Apply Zemits RevitaOx Pro serum to infuse the skin with antioxidants and Hyaluronic Acid. This serum hydrates the skin, soothes any irritation from acne, and provides antioxidant protection to prevent further damage and promote healing.

4. Cold Therapy:

- Application of Zemits FirmingGenius Pro: Apply Zemits FirmingGenius Pro gel and use a Cold Wand to gently massage the skin. Cold therapy helps to calm the skin, reduce any redness, tighten pores, and lock in hydration, leaving the skin feeling refreshed and balanced.

5. Post-Treatment Care:

- Advise the teen on proper skincare routines, including gentle cleansing, moisturizing, and the importance of sunscreen to protect their skin from UV damage.
- Recommend products that are suitable for their skin type, such as non-comedogenic moisturizers and oil-free sunscreen, to help maintain clear and healthy skin between treatments.

HydroDiamond DePuff

De-Puffing & Detox Treatment Protocol

Products to Use:

- Zemits SalicPure Pro Purifying HydroDermabrasion Serum with Salicylic Acid
- Zemits RevitaOx Pro Hydro-Revival Antioxidant Infusion Serum with Hyaluronic Acid
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides

Series of Treatments:

4-6 treatments recommended, spaced 2-4 weeks apart



ZEMITS SALICPURE PRO

- Unclogs pores
- Purifies and detoxifies
- Reduces inflammation
- Gently exfoliates
- Improves skin clarity



ZEMITS REVITAOX PRO

- Hydrates skin
- Provides antioxidant protection
- Reduces puffiness
- Soothes and calms
- Enhances detoxification



ZEMITS FIRMINGGENIUS PRO

- Firms and tightens skin
- Reduces fluid retention
- Soothes and cools
- Enhances skin elasticity
- Locks in hydration

STEPS

1. Pre-Treatment Preparation:

- Ensure the treatment area is clean and equipped with all necessary tools and products.
- Conduct a consultation with the client to assess their concerns regarding puffiness, fluid retention, and signs of fatigue. Identify any areas of the face or body that require focused attention.

2. Cleansing and Exfoliation:

- Cleansing: Begin with a gentle cleanse to remove any impurities, excess oil, and makeup from the skin. Use a cleanser suitable for the client's skin type to ensure a fresh start to the treatment.
- Application of Zemits SalicPure Pro: Apply 2-3 ml of Zemits SalicPure Pro serum to exfoliate the skin. This serum helps to purify the skin, unclog pores, and promote detoxification by removing dead skin cells and promoting circulation.
- Exfoliation with HydroDiamond: Use the HydroDiamond wand with a suitable tip to gently exfoliate the skin, focusing on areas prone to puffiness and fluid retention, such as under the eyes, cheeks, and jawline. This step helps to refine the skin texture and prepare it for the detoxifying and de-puffing treatments.

3. Oxygen Serum Infusion:

- Application of Zemits RevitaOx Pro: Apply Zemits RevitaOx Pro serum, which is rich in antioxidants and Hyaluronic Acid. This serum helps to hydrate the skin, reduce inflammation, and promote lymphatic drainage, making it effective for detoxification and reducing puffiness.
- Oxygen Infusion: Use an oxygen infusion device to help the serum penetrate deeper into the skin, enhancing its detoxifying and de-puffing effects.

4. Cold Therapy:

- Application of Zemits FirmingGenius Pro: Apply Zemits FirmingGenius Pro gel and use a Cold Wand to gently massage the skin. Cold therapy is crucial in this protocol as it helps to constrict blood vessels, reduce fluid retention, and soothe the skin. Focus on areas with visible puffiness to enhance the de-puffing effect.

5. Post-Treatment Care:

- Advise the client on post-treatment care, including the use of a lightweight moisturizer and sunscreen. Emphasize the importance of staying hydrated and maintaining a healthy diet to support the detox process.
- Recommend a series of 4-6 treatments for optimal results, depending on the client's needs and desired outcomes.

HydroDiamond GlowMama

Safe & Soothing Facial Treatment Protocol

The GlowMama Facial Treatment Protocol

Is specially designed to cater to the unique skincare needs of expectant mothers. This gentle, nourishing treatment focuses on hydration, calming sensitive skin, and enhancing the natural pregnancy glow while ensuring absolute safety for both the mother and baby.



Products to Use:

- Zemits HydroClear Pro Clarifying HydroDermabrasion Serum with Lactic Acid
- Zemits RevitaOx Pro Hydro-Revival Antioxidant Infusion Serum with Hyaluronic Acid
- Zemits FirmingGenius Pro Collagen Boosting Conductive Gel with Peptides

Series of Treatments:

Recommended every 4-6 weeks throughout pregnancy.



ZEMITS HYDROCLEAR PRO

- Gently exfoliates
- Smooths skin texture
- Reduces dryness
- Safe for sensitive skin
- Prepares skin for hydration



ZEMITS REVITAOX PRO

- Deeply hydrates
- Soothes irritation
- Provides antioxidant protection
- Enhances skin glow
- Supports skin health



ZEMITS FIRMINGGENIUS PRO

- Calms and soothes skin
- Reduces redness
- Locks in moisture
- Firms and tightens
- Provides a cooling effect

STEPS

1. Pre-Treatment Preparation:

- Ensure the treatment area is clean, comfortable, and relaxing.
- Conduct a detailed consultation with the client to understand any specific skin concerns related to pregnancy, such as sensitivity, dryness, or hormonal changes.
- Use bolsters and pillows to provide extra support and comfort during the treatment.

2. Gentle Cleansing:

- Begin with a mild, pregnancy-safe cleanser to remove impurities and prep the skin for treatment.
- Use gentle, circular motions to avoid overstimulation of sensitive skin.

3. Mild Exfoliation:

- Application of Zemits HydroClear Pro: Apply a small amount of Zemits HydroClear Pro serum. This gentle exfoliant with Lactic Acid is safe for pregnancy and helps remove dead skin cells without causing irritation.
- Light Exfoliation: Use a very gentle exfoliating technique, avoiding any aggressive pressure. Focus on areas prone to dryness or dullness.

4. Hydration and Oxygen Infusion:

- Application of Zemits RevitaOx Pro: Apply Zemits RevitaOx Pro serum to infuse the skin with hydrating and antioxidant-rich ingredients. This serum helps soothe and hydrate the skin, making it ideal for the sensitive skin of expectant mothers.
- Oxygen Infusion: Use an oxygen infusion device to enhance the serum's absorption, ensuring the skin is deeply hydrated and revitalized.

5. Soothing Cold Therapy:

- Application of Zemits FirmingGenius Pro: Apply Zemits FirmingGenius Pro gel and gently massage the skin using cold therapy. The cooling effect helps to calm any redness or inflammation, providing relief to sensitive skin.
- Focus on areas where the skin may be more reactive, ensuring a soothing and comforting experience.

6. Relaxing Facial Massage:

- Perform a gentle facial massage using pregnancy-safe oils or creams. This step enhances relaxation, reduces tension, and promotes the natural pregnancy glow.
- Avoid any pressure points that may be sensitive during pregnancy.


7. Post-Treatment Care:

- Advise the client on pregnancy-safe skincare routines, including the use of gentle, hydrating products and sunscreen to protect the skin.
- Recommend regular treatments every 4-6 weeks to maintain skin health and support the natural glow throughout pregnancy.



WATCH AND LEARN:

Step-by-Step HydroDiamond Facial Training

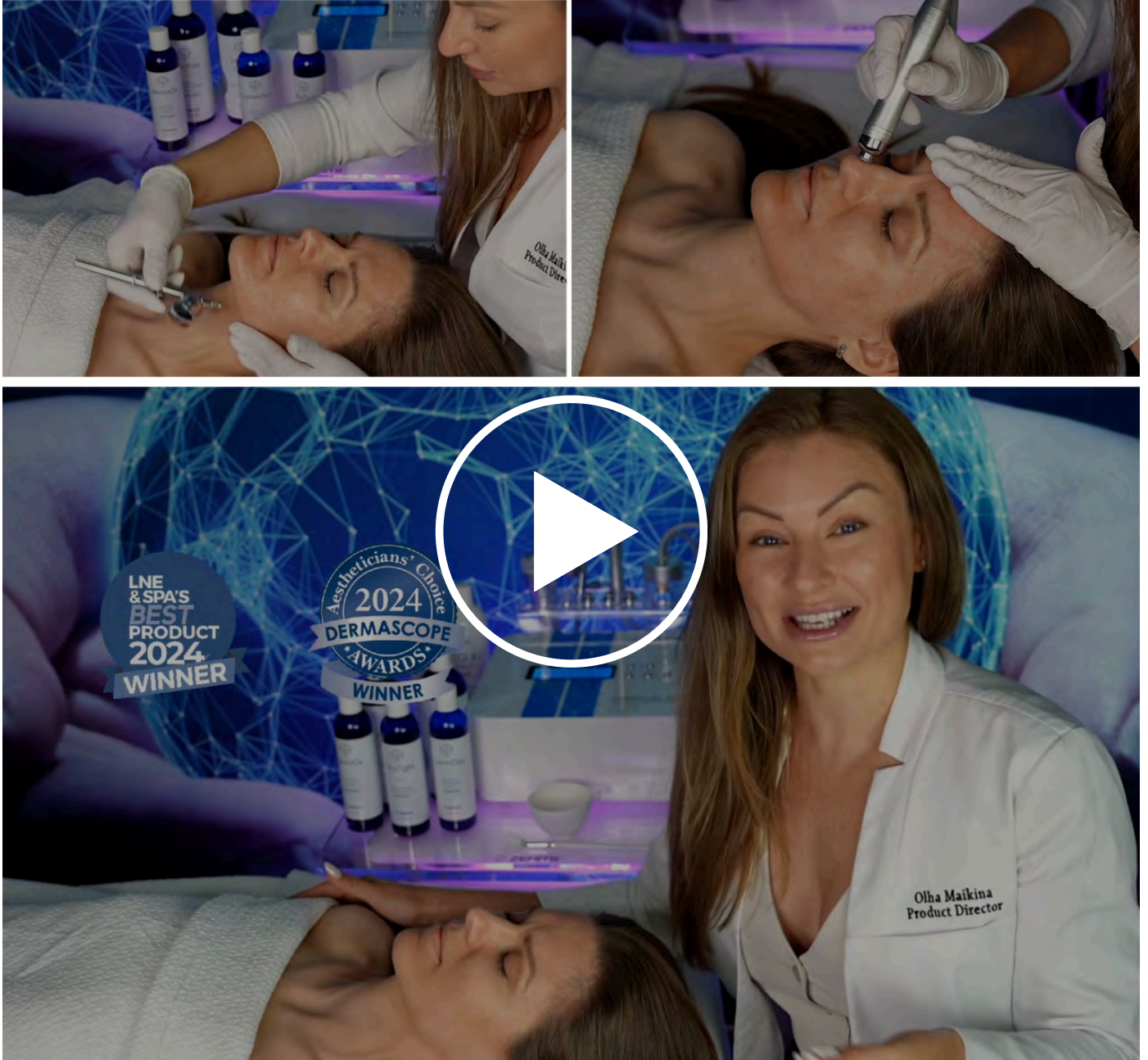

WATCH NOW

