

# DERMA CHANNEL NANO INFUSION PEN

TOOL

## Resurface & Infuse



**SOUND BITE:** *"Shake up your treatments"*

Non-invasive, nano infusion pen with a mono-crystalline silicon cartridge creates microscopic channels in the outermost layer of the epidermis to resurface skin and boost the absorption and efficacy of topical skincare products.

### WHAT DOES IT DO?

- Visibly reduces lines and hyperpigmentation
- Helps improve skin tone and texture
- Boosts radiance and elasticity
- Optimizes absorption of topical skincare

### DIRECTIONS

After cleansing and prepping face, apply serum. Press and hold power button to turn on unit. Press button to increase intensity of strokes (Level 1 = blue, Level 2 = green, Level 3 = red). Perform treatment by moving unit over each section in cross-hatched pattern. For more details, visit [www.lemieuxpro.com](http://www.lemieuxpro.com)

### SKIN TYPES

All skin types

### CONDITIONS

Lines and wrinkles, dullness, uneven tone and texture, hyperpigmentation

### POST TREATMENT

Do not apply makeup for at least 30 minutes after using unit. Avoid activities that increase core body temperature at least 24 hours after treatment. Do not use exfoliants for at least 3 days after treatment. Avoid extended sun exposure for at least 3 days after treatment. Always use mineral-based sunscreen after treatment.

### CAUTION

Keep out of reach of children. Disposable needle cartridges are for single use only. Operate this unit only according to its intended use as described in User Guide. Do not use this unit on irritated or broken skin. Do not use this unit on keloids, moles, warts, raised scarring, lesions, or during a flare-up of eczema, psoriasis, or other inflammatory skin conditions. Do not use this unit on eyelids or over bone in the periorbital region.