

BioSculpt7 LED Mask Neck & Décolleté Treatment

A rejuvenating LED treatment designed to visibly improve the appearance of the neck and décolleté through targeted light therapy. This protocol combines advanced LED technology with professional growth factors, cryotherapy, and thermal Gua Sha massage to support firmer, brighter, and healthier-looking skin.



KEY BENEFITS

- ✓ Promotes healthier-looking neck & décolleté skin
- ✓ Supports healthy collagen production
- ✓ Improves the appearance of firmness & smoothness

SKIN TYPE: All Skin Types

DURATION: 45 minutes

COST PER TREATMENT: \$3.34

SUGGESTED MENU PRICE: \$200-\$250

CONTRAINDICATIONS:

Broken, irritated, or inflamed skin / Recent professional treatments or cosmetic injectables / Pregnancy (consult physician) / Pacemaker or implanted electronic devices / Photosensitizing medications / Epilepsy / Consult device manufacturer safety guidelines before treatment.

STEP 1: CLEANSE I

- Perform first cleanse with 1-2 pumps of **PurErb Bare Beauty Cleansing Oil**.
- Apply to dry skin with dry hands and complete cleanse routine. Add water to emulsify and remove.

STEP 2: SKIN ANALYSIS

- Identify and confirm client skin concerns and assess skin barrier integrity to determine product customization.

STEP 3: CLEANSE II

- Combine pea-sized amount of **Peptide Foam Cleanser** with one pump of **O² Calming Gel** in a cup.
- Whip into a foam consistency with a fan brush and apply to face, neck, and décolleté, avoiding eye area.
- Perform cleanse routine and remove thoroughly with wet cotton or sponges.

STEP 4: PREP FOR LED LIGHT THERAPY

- Saturate cotton round with **Iso-Cell Recovery Solution** and swipe over neck, and décolleté to remove any residue.
- Allow neck and décolleté to completely dry before LED mask application.
- **IMPORTANT:** For optimal LED light penetration, the neck and décolleté must be: clean, completely dry, free of serums, moisturizers, SPF or makeup. **Moisture or topical products on the skin may interfere with proper contact and light delivery.**

STEP 5: BIOSCUPT7 LED THERAPY NECK & DÉCOLLETÉ

- Position the **BioSculpt7 LED Neck & Décolleté Mask** on the neck and décolleté.
- Select the targeted light therapy program based on the client's skin concerns. (See **BioSculpt7 LED Neck & Décolleté Mask** treatment guide for customization.)
- Allow treatment to run for 10-15 minutes, while continuing to the next step. Ensure the mask maintains full contact with clean, dry skin throughout the treatment.

STEP 6: GUA SHA THERMALIFT FACE

- Apply 2-3 drops of any Le Mieux serum followed by a layer (1-2 pumps) of **O² Calming Gel** to the face.
- Turn on the **Gua Sha Thermalift** to INFUSION mode and perform gua sha strokes along meridian facial lines for the duration of **BioSculpt7 LED Neck & Décolleté** treatment.
- Remove the **BioSculpt7 LED Neck & Décolleté Mask** after the full treatment cycle.

STEP 7: GUA SHA THERMALIFT NECK & DÉCOLLETÉ

- Apply **Cryo Peptide Eye Mask** along the orbital bone, avoiding direct contact with the lash line. Leave on while continuing the remainder of the treatment.
- Apply 2-3 drops of **StemCell EGF** followed by a pea-sized amount of **Midnight Balm** to neck & décolleté.
- Turn on the **Gua Sha Thermalift** to INFUSION mode and perform gua sha strokes along meridian neck & décolleté lines.

STEP 8: FINISH

- Remove the **Cryo Peptide Eye Mask**.
- Apply 2-3 drops of any Le Mieux Serum and a pea-sized amount of **Moisture Cloud Rescue Cream** or preferred Le Mieux moisturizer to the face, neck, and décolleté.
- Finish with **Kizo Lab UV Defense Clear Sunscreen SPF 50+** on face, neck, and décolleté.



Scan to watch protocol video.