

DERMA CHANNEL NANO INFUSION PEN



Non-invasive nano-infusion technology gently resurfaces the epidermis, enhancing the absorption of Le Mieux serums for visibly smoother, rejuvenated skin and improved product efficacy. Perfect for targeting fine lines, texture, and hydration.

BENEFITS

Improves serum saturation & product penetration
Refines texture
Hydrates
Refines Pores

SKIN TYPE:

All skin

CONTRAINDICATIONS:

Check Le Mieux
Contraindication Guide

COST PER TREATMENT:

\$6.72

SUGGESTED MENU PRICE:

\$150-250

PROTOCOL SUPPLIES

- PurErb Bare Beauty Cleansing Oil
- Peptide Foam Cleanser
- Ionized Oxygen Infuser
- Iso-Cell Recovery Solution
- EGF-DNA
- Hyaluronic Serum
- Derma Channel Nano Infusion Pen
- Bio Cell+ Mask
- O2 Calming Gel
- Bio Cell Rejuvenating Cream
- Just Glow BB Cream SPF 50

SUGGESTED HOME CARE

- Peptide Foam Cleanser
- Iso-Cell Recovery Solution
- EGF-DNA
- Hyaluronic Serum
- Derma Relief Serum
- ProFerm Face Glow Mask
- Just Glow BB Cream SPF 50

CLEANSE I

Perform first cleanse with 1-2 pumps of **PurErb Bare Beauty Cleansing Oil**. Apply to dry skin with dry hands and complete cleanse routine. Add water to emulsify and remove.

ANALYSIS

Identify and confirm client skin concerns. Assess skin barrier integrity. Do not perform passes over areas of inflammation, broken barrier, or post-inflammatory hyperpigmentation.

CLEANSE II

Lather a dime-size amount of **Peptide Foam Cleanser** with water. Perform second cleanse and remove.

SERUM INFUSION

Apply customized serum blend in **Ionized Oxygen Infuser**.

- Load mini-flask with **Iso-Cell Recovery Solution**.
- Add 2 drops of **EGF-DNA** for normal to dry, non-breakout prone skin.
- Add 2 drops of **Hyaluronic Serum** for normal to oily, breakout-prone skin.

DERMA CHANNEL NANO INFUSION PEN

- Attach single-use nano cartridge.
- Hold power button to turn on.
- Press power button to increase intensity (RPM)
- Apply **Hyaluronic Serum** to working section.
- Hold skin taut as you work (See Pass Guide).
- Begin at chin or jawline.
- Complete passes by section no larger than a postage stamp.
- DO NOT APPLY PRESSURE.
- Avoid inner eyelid areas.
- Gauge client sensations. Adjust tool intensity as needed.

MASK

Apply **Bio Cell+ Sheet Mask**. Process for 10 minutes. Do not rinse. Apply **O2 Calming Gel** to areas of redness.

HYDRATE & PROTECT


Apply custom Serum Infusion blend.

Apply **Bio Cell Rejuvenating Cream**: peptide-infused moisture for normal to dry skin.

Finish with **Just Glow BB Cream SPF 50**.

PASS GUIDE


VERTICAL
PASS

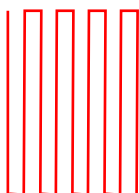

HORIZONTAL
PASS




COMPLETE SECTION

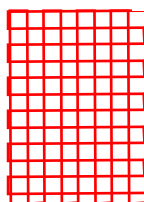

Cross Hatch Passes


- Work in 1 x 1 inch sections.
- Hold section taut between thumb and middle finger.
- Saturate section with **Hyaluronic Serum**.
- Make vertical serpentine passes in a 1x1 inch section
- Make horizontal serpentine passes in the same 1x1 inch section.
- **DO NOT APPLY PRESSURE.**
- Client should not feel uncomfortable sensations.
- Turn head as needed.

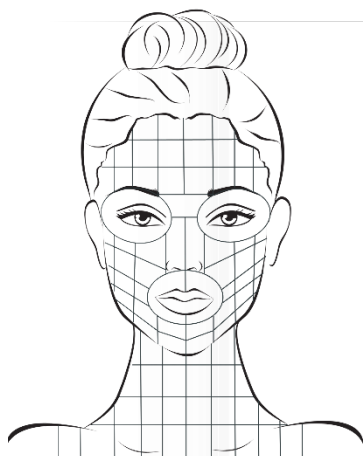

Passes: Face, Neck & Chest

Complete passes by section, avoiding eyelid areas.


- Begin at jaw, working up to forehead.
- Perform crosshatch passes.
- Avoid inner eyelid areas.
- Gauge client sensations.
- Adjust tool intensity as needed.

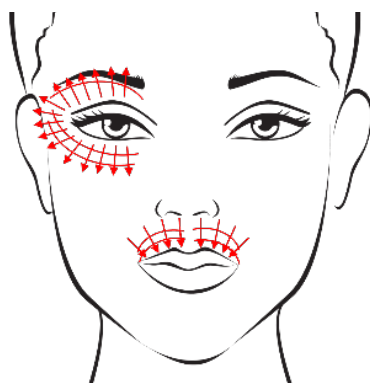

Passes: Eye + Lip

Periorbital & perioral areas


- Use Level 1 (blue) Intensity only.
- Perform cross hatch passes
- Work away from eyes.
- Work inward over lips.
- **DO NOT APPLY PRESSURE.**
- Do not work directly over eyes.
- Can be used over vermillion line.

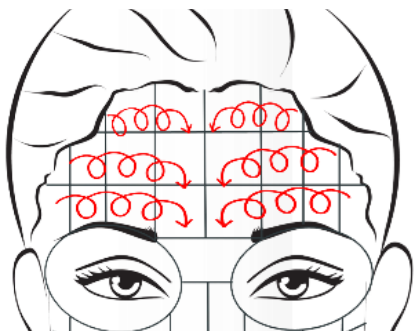

Passes: Forehead

Above brows, up to hair line

- Saturate section with **Hyaluronic Serum**.
- Hold section taut.
- Perform fluid circular passes working horizontally from brow to hairline.
- **DO NOT APPLY PRESSURE.**
- Gauge client sensations.
- Adjust tool intensity as needed.

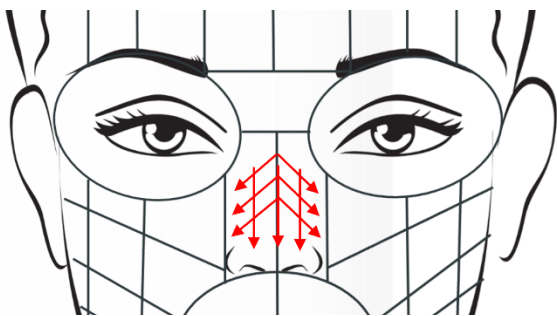

Passes: Nose

Below corrugator to nostrils

- Divide into 2 sections: left and right.
- Complete passes running downward, parallel with bridge.
- Complete passes running outward from bridge to cheek.
- Turn nose by tip to hold sections taut.
- Work away from eyes.
- **DO NOT APPLY PRESSURE.**