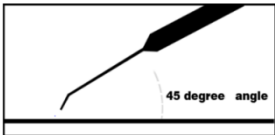


SKIN FIRING FACIAL with RF & Cryo-Electrophoresis Machines

*Radio frequency and cryo-electrophoresis machines enhance this skin tightening and rejuvenating facial. Features ultrasonic and gua sha treatments to naturally **detoxify skin, improve texture and tone, minimize fine lines and wrinkles, and lift facial contours.***

PRODUCTS		TIME	AREA
<ul style="list-style-type: none"> • Phyto-Marine Cleansing Lotion • M15 (15% Mandelic + Arbutin Peel) • Gua Sha Tool (wind, crown, or cup) • Ultrasonic Facial Scrubber • Iso-Cell Recovery Solution • Hyaluronic Serum • Vita-C Serum • TGF-β Booster • Hyaluronic Shea Mask 	<ul style="list-style-type: none"> • EGF-DNA • Moisture Infusion Mask • Eye Wrinkle Corrector • Bio Cell Rejuvenating Cream • Vita-C Skin Protective Crème • Pomegranate Massage Cream (optional) • Collagen Peptide Serum (optional) 	50-65 min	Face, Neck, Décolleté
OTHER INFORMATION			
Product Investment: \$7 - \$8 Facial Price: \$100 - \$130 Home Care: TGF-β Booster Eye Wrinkle Corrector EGF-DNA Vita-C Serum			

PROCEDURE

- Cleanse I** **3 min** Cleanse the facial area with **Phyto-Marine Cleansing Lotion**. Remove with a warm towel or moistened sponge. Pat dry.
- Exfoliate I** **1-2 min** PROTECT EYE AREAS with moistened cotton pads. Use gauze pad to apply 1-2 layers of **M15 (15% Mandelic + Arbutin Peel Solution)** on face, neck, and décolleté.
- Massage** **5 min** Perform scalp massage with **gua sha tool**.
- Exfoliate II** **5 min**  Glide the ultrasonic **Facial Scrubber** with the tip of the spatula angled downward across the face. Keep skin moistened with **Iso-Cell Recovery Solution** during exfoliation.
- Serum** **2-3 min** Massage 3-4 drops of **Hyaluronic Serum, Vita-C Serum, and TGF-β Booster**, layering serums in that order onto face, neck, and décolleté.
- Mask** **5** Apply **Hyaluronic Shea Mask** evenly with a fan/facial brush. Massage into skin.
- Machine** **20 min** Turn **RF machine** setting to **40-46**. Adjust to client's comfort. Massage skin for **10 minutes** each side.
- Serum** **2-3 min** Massage 3-4 drops of **EGF-DNA** onto face, neck, and décolleté.

-
- 9 **Cold Treatment** **5-7 min** Apply **Moisture Infusion Mask** onto face. Use grounding pole to ground client. Set **CEP machine** to 10° C and glide wand over mask for 5 minutes.
-
- 10 **Hot Treatment** **5-7 min** *OPTIONAL:* While mask is still on, apply **Pomegranate Massage Cream** to décolleté. Set **CEP machine** to 40° C and use wand to massage cream into skin for 5 minutes. Remove massage cream with a moistened sponge or warm towel.
-
- 11 **Eyes & Lips** **1 min** Remove **Moisture Infusion Mask**. Do not rinse. Gently pat **Eye Wrinkle Corrector** around eyes.
-
- 12 **Hydrate** **1 min** Massage **Bio Cell Rejuvenating Cream** or **Collagen Peptide Serum** (*optional*) over face, neck, and décolleté.
-
- 13 **Sun Care** **1 min** Complete procedure with **Vita-C Skin Protective Crème**.