

THE ULTIMATE FACIAL with Ultrasonic Facial Scrubber

Ultrasonic technology enhances this rejuvenating facial that smooths fine lines and wrinkles, deeply hydrates, and restores skin radiance.

PRODUCTS	TIME	AREA
<ul style="list-style-type: none"> • Phyto-Marine Cleansing Lotion • Iso-Cell Recovery Solution • Hungarian Enzyme Mask • Vita-C Clear Skin Pad • 5% Glyco Polymer Solution (G5) • Pomegranate Massage Cream • Essence Toner • TGF-β Booster • Bio Cell+ Mask 	45 - 60 min	Face, Neck, Décolleté
OTHER INFORMATION		
Product Investment: \$5 - \$6 Facial Price: \$80 - \$100 Home Care: <ul style="list-style-type: none"> • TGF-β Booster • EGF-DNA • Eye Wrinkle Corrector • Collagen Peptide Serum 		


PROCEDURE

- 1 **Cleanse** 3-5 min Cleanse the facial area with **Phyto-Marine Cleansing Lotion**. Remove with a warm towel or moistened sponge. Pat dry.

Option:
clarify acne skin

*Option: For acne skin, use **Exfoliating Cleansing Gel** instead.*

- 2 **Exfoliate I** 1-2 min

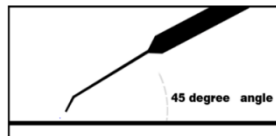


Slide the **Facial Scrubber** with the tip of the spatula angled downward across the face. Keep skin moistened with **Iso-Cell Recovery Solution** during exfoliation.

- 3 **Steam** 5 min Combine ½-1 scoop **Hungarian Enzyme Mask** with 2-3 sprays of **Iso-Cell Recovery Solution**. Steam for approximately 2-3 minutes. Remove with a moistened sponge.

- 4 **Exfoliate II** 2-3 min **PROTECT EYE AREAS** with moistened cotton pads. Use 1-2 **Vita-C Clear Skin Pads** and gently perform 2-3 passes on face, neck, and décolleté. Do not rinse. On the same **Vita-C Clear Skin Pad**, add 2-3 drops of **5% Glyco Polymer Solution (G5)**. Perform 2-3 passes. Do not rinse.


- 5 **Extract** 8-10 min

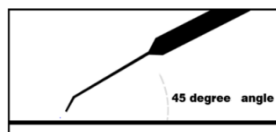


Slide **Facial Scrubber** angled downward where extraction is needed. Keep skin moistened with **Iso-Cell Recovery Solution**. After extraction, mist **Iso-Cell Recovery Solution** onto the palm of your hand and pat onto client's face to soothe skin irritation.

6 **Massage** 8-10 min Perform massage with **Pomegranate Massage Cream**. Massage over face, neck, and décolleté for 8-10 minutes. Remove with moistened sponge or warm towel.

7 **Tone** 1 min Mist **Essence Toner** 2-3 times directly into the palm of your hand and pat onto client's face.

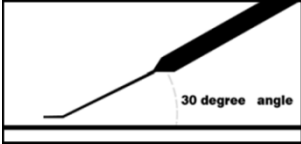8 **Serum** 2-4 min  Massage 3-4 drops of **TGF-β Booster** onto the face, neck, and décolleté in an upward and outward motion. Infuse serum by gliding **Facial Scrubber** with tip of spatula flat against skin.

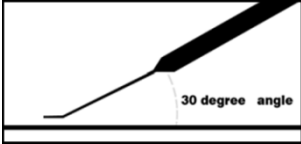9 **Mask** 15-20 min  Apply **Bio Cell+ Mask**. Apply the middle freeze-dried collagen layer and discard the 2 outer protective layers (the paper-like layer and the protective gauze layer). Spread the mask evenly onto the client's face. Glide the **Facial Scrubber** over **Bio Cell+ Mask** with tip of spatula flat against skin. Remove mask after 10-15 minutes. Do not rinse after mask application.

10 **Eye & Lip Care** 1 min Gently pat **Eye Wrinkle Corrector** around eyes.

Option:
treat deep lines
around eyes

*Option: For clients with deeper lines, apply **Eye & Lip Cream** over **Eye Wrinkle Corrector** around the eyes.*

11 **Hydrate** 1 min Massage **Bio Cell Rejuvenating Cream** over face, neck, and décolleté.

12 **Sun Care** 1 min Complete procedure with **Vita-C Skin Protective Crème**.

Option:
sun care for
acne skin

*Option: For acne skin, use **Skin Protective Emulsion** instead.*