

## Clear Complexion Care with DermaBlu LED

*This skin-perfecting treatment is designed to improve the appearance of blemish-prone skin and breakouts, promoting healthier-looking skin and a clearer complexion.  
Specially customized for **OILY, ACNE-PRONE SKIN.***

PRODUCTS		TIME	FACIAL PRICE
<ul style="list-style-type: none"> <li>• Exfoliating Cleansing Gel</li> <li>• Hungarian Enzyme Mask</li> <li>• Iso-Cell Recovery Solution</li> <li>• O2 Calming Gel</li> <li>• Moisture Infusion Mask</li> <li>• Hyaluronic Serum</li> <li>• Eye Wrinkle Corrector</li> <li>• Skin Protective Emulsion</li> </ul>	<b>Tool:</b> <ul style="list-style-type: none"> <li>• DermaBlu LED</li> </ul>	40 - 60 min	\$85 - \$125
<h3>HOME CARE</h3> <ul style="list-style-type: none"> <li>• Exfoliating Cleansing Gel</li> <li>• Essence Toner</li> <li>• Iso Cell Recovery Solution</li> <li>• O2 Calming Gel</li> <li>• Hyaluronic Serum</li> <li>• Eye Wrinkle Corrector</li> </ul>			

### PROCEDURE

- Cleanse**      **4-5 min**      Apply small amount of **Exfoliating Cleansing Gel** on face, neck, and décolleté, avoiding contact with eyes.  
Turn on **DermaBlu**, then press **ION+** button 1 time for low intensity (steady green light) or 2 times for high intensity (flashing green light).  
**ION+**  
Move the ultrasonic head in slow, circular movements. Do not use near eyelids. Remove remaining gel with warm water or moistened towel.
- Exfoliate**      **5-6 min**      Combine 1/2 - 1 scoop **Hungarian Enzyme Mask** with **Iso-Cell Recovery Solution** so it has consistency of thin icing. Steam for 5 minutes. Remove with moistened sponge or towel.
- Extract**      **5-8 min**      Perform extractions as needed. Pat **Iso-Cell Recovery Solution** onto targeted areas to calm skin irritation.
- DermaBlu LED**      **5-10 min**      Apply **O2 Calming Gel**. Turn on **DermaBlu**, then press **LED** button and **SONIC** buttons 1 time (steady green light). Move ultrasonic head in slow, circular movements over targeted areas.  
**LED**      **SONIC**

---

5	<b>Hydrate</b>	<b>15-20 min</b>	Apply <b>Moisture Infusion Mask</b> . Remove after 15-20 minutes. Do not rinse.
6	<b>Serum</b>	<b>5-10 min</b>	Apply 2-3 drops <b>Hyaluronic Serum</b> onto face, neck, and décolleté.
8	<b>Eye Treatment</b>	<b>1 min</b>	Gently pat <b>Eye Wrinkle Corrector</b> around eyes.
9	<b>Sun Care</b>	<b>1 min</b>	Complete procedure with <b>Skin Protective Emulsion</b> .

---